septodont	SEPTODONT ARTWORK CODE: M05 93 023 20 00	COLOR(S)	3	NOT PRINTABLE - AREA FOR VARIABLE DATA STICKER
	CREATION DATE: 9/07/24	Pantone 182	Pantone 427	
	MODIFICATION DATE: 10/07/24			The following information will be printed on line during manufacturing process :
	DIMENSIONS : 156 mm x 125 mm			LOT + variable data text + variable data text UDI QR code + variable data text
GRAPHIC DESIGNER: Jean				

REF 245595

Parcan L

3% stabilised ready to use solution
of purified sodium hypochlorite
for root canal irrigation.

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en - Description : Parcan L is an aqueous sodium hypochlorite solution. At low concentrations dissolve mainly necrotic tissue, at higher concentrations also dissolves vital tissue. **Procedure:** It is used for the irrigation and debridement of root canals during endodontic treatment. Remove smear layer with appropriate irrigant such as EDTA solution or Citric Acid 12.5%. Only for professional use. **Instructions for use:** 1. Remove the gasket. 2. Fill syringe with Parcan L (10 ml recommended). 3. Attach irrigation tip securely to syringe. 4. Irrigate as desired. 5. Remove and dispose of syringe and irrigating tip. **Precautions:** 1. Read MSDS for material handling and safety information. 2. Irrigation should not be irritant for the apical area. 3. Avoid spilling on surrounding tissues. 4. Avoid spilling on colored fabrics and surfaces as Parcan L is moderately alkaline and has a whitening effect. **Warnings:** 1. Caution, caustic! 2. Keep out of the reach of children. 3. Avoid contact with skin and eyes. 4. Do not swallow or inject. 5. Avoid mixing Parcan L with CHX. **Storage:** Store in a dry area away from heat and direct sunlight. **Date of issue:** 06-2023

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